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### Instruction manual - sanowell adhesive buffers for walking aids

Please read this user manual carefully prior to first use!

#### Cleaning your adhesive buffers

Stains and soiling may be removed with a soft cloth and a mild detergent. Rinse well with clear water and allow to dry thoroughly. Do not use detergents containing solvents as these may cause damage to the material. Never clean the adhesive buffers with oily substances. Risk of slipping!

#### Adhesive buffers may only be mounted or demounted by qualified persons. Please contact your specialist retailer!

#### Technical details anti-slip adhesive buffers for walking aids:

Item no. 89316 - for walking aid tube with a 16 mm Ø  
Item no. 89318 - for walking aid tube with a 18 mm Ø  
Item no. 89320 - for walking aid tube with a 20 mm Ø  
Item no. 89322 - for walking aid tube with a 22 mm Ø

Height of the adhesive buffer: approx. 74,5 mm  
Diameter of the contact surface: approx. 39,5 mm  
Weight: approx. 65 g  
Maximum stability under load: 100 kg

#### Application and safety notices:

Prior to each use, check the functionality of your walking aid and ensure the tight fit of the sanowell adhesive buffers. Regularly check the adhesive buffers for damage or wear and replace them if necessary. In case of heavy strain, replace your sanowell adhesive buffers approx. every 3-4 months. For safety reasons, the top and the bottom part were joined mechanically. If the bottom part is worn down, it may not be detached from the top part and replaced separately. The bottom part may become darker over time due to incident light radiation. This does not impair the quality in any way. The top part is highly resistant against ageing. The danger of slipping or falling down - especially on slippery or wet surfaces - can never entirely be ruled when using walking aids. Place the walking aids as vertical as possible on the surface when walking and not too far away from the body. Setting the walking aid down at a very slanted angle increases the risk of falls!

#### Design of the sanowell adhesive buffers:

Two-part structure consisting of a soft bottom part made of elastomer and a hard top part made of rubber with an integrated metal disk to prevent the walking aid tube from breaking through.

#### Function and purpose of the sanowell adhesive buffers:

The anti-slip, low-abrasion sanowell adhesive buffers are mounted on walking aids with an external diameter of 16, 18, 20 or 22 mm and give the user a lot of stability, especially on wet surfaces. The two-part structure of the sanowell adhesive buffers cushions blows and allows safe standing with the joint plug, even if the walking aid is set down slightly slanted.

#### Storage notice:

The sanowell adhesive buffers should be stored in a cool and dry place away from direct sunlight.

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